



CE
2292

SURGICLEAN[®]

**Absorbable Gauze of
Oxidized
Regenerated
Cellulose**

Control Bleeding Improve Healing

What is Surgiclean® Absorbable Gauze of Oxidized Regenerated Cellulose?

It's medical gauze for surgery.

The raw material is regenerated cellulose made from specialty fiber through oxidation and trimming.

This smart hemostatic material can be decomposed and absorbed during the healing process of human wound tissue.

As a supporting hemostatic device, Surgiclean® can be used for various surgical procedures.

It is especially ideal in capillary, vein and arteries where the traditional hemostasis is limited such as ligation. At the same time, it can be used instead of ordinary gauze during surgery to stop bleeding as well as reduce the possibility of postoperative adhesions.

● How it works

Step1


Blood pass through the gauze slowly

Step2

Gauze absorb the blood and forms into a tight physical seal at the bleeding surface

Step3

Hemostasis within 3 minutes, fully absorbed with 7-30 days.

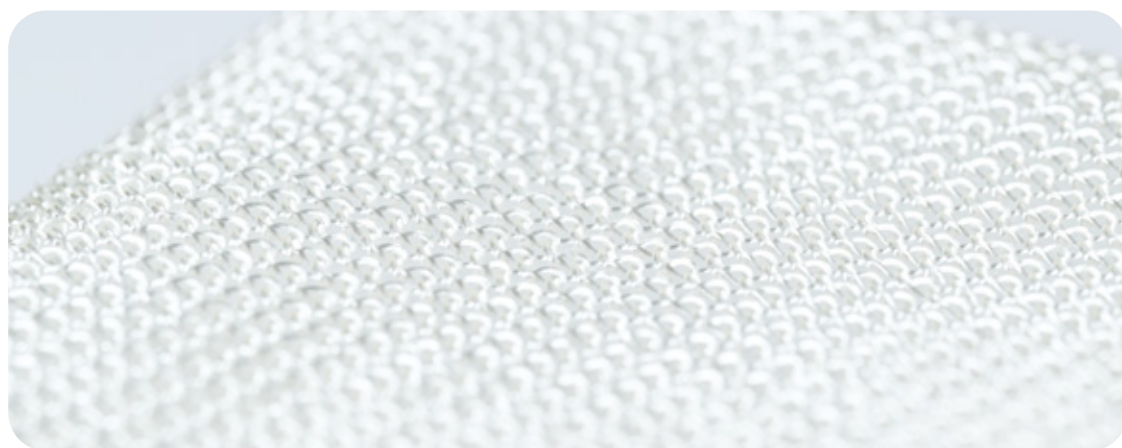

Storage

It should be stored in clean, dry, ventilated and non-corrosive environment, and the temperature should be no more than 20℃.

Brand	Specification
	2cm x 3cm/pc
	3cm x 5cm/pc
	5cm x 5cm/pc
	5cm x 7.5cm/pc
	5cm x 8cm/pc
	5cm x 10cm/pc
	7cm x 8cm/pc
	5cm x 25cm/pc
	5cm x 30cm/pc
	5cm x 35cm/pc
	10cm x 10cm/pc
	10cm x 15cm/pc
	10cm x 20cm/pc

It can be customizable

Application

Applicable Departments:

General Surgery / Nail Surgery / Hepatobiliary And
Pancreatic Surgery / Gastrointestinal Surgery / Gynecology

Various Open And Endoscopic Surgery To Stop Bleeding

Applicable Surgery Type: Appendectomy / Breast Surgery /
Hernia Repair / Gastrectomy / Hepatectomy / Hepatobiliary
Bladder Surgery Gynecological Surgery, Thyroid Surgery,
Treatment Of Superficial Injuries, Etc.

Ready to use, right out of the box



WHY?

SURGICLEAN®

Ordinary look with extraoRdinary performanCe

Plant-based material

Low PH creates bactericidal environment

Multiple sizes/shapes

Sterile and Biodegradable

Hemostasis within 3 minutes

Absorbed with 7-30 days

CE
2292

Hangzhou Singclean Medical Products Co., Ltd.

Add: No.125(E),10th Street, Hangzhou Qiantang Area, Zhejiang, China

Tel: +86-571-63431868

WhatsApp: +8613362195396

E-mail: marketing@singclean.net

P.c.: 310018

Fax: +86-571-63431866

www.singclean.net

More information on



Model No.

Product	Specification	Model No. Surgiclean®
Absorbable Gauze of Oxidized Regenerated Cellulose	2cm*3cm/pc	ORC-SG-2×3
	2cm*5cm/pc	ORC-SG-2×5
	3cm*5cm/pc	ORC-SG-3×5
	5cm*5cm/pc	ORC-SG-5×5
	5cm*7.5cm/pc	ORC-SG-5×7.5
	5cm*8cm/pc	ORC-SG-5×8
	5cm*10cm/pc	ORC-SG-5×10
	7cm*8cm/pc	ORC-SG-7×8
	10cm*10cm/pc	ORC-SG-10×10
	10cm*15cm/pc	ORC-SG-10×15
	10cm*20cm/pc	ORC-SG-10×20
	5cm*25cm/pc	ORC-SG-5×25
	5cm*30cm/pc	ORC-SG-5×30
	5cm*35cm/pc	ORC-SG-5×35
	22.5cm*22.5cm/pc	ORC-SG-22.5×22.5
	30cm*30cm/pc	ORC-SG-30×30
	30cm*50cm/pc	ORC-SG-30×50